

Grade

RADIOLOGY EQUIPMENT FLUOROSCOPIC ROOM CHECK-OFF

Student: _____ Clinical Affiliate: _____ Date: _____
 Evaluator: _____ Equipment room number: _____

EVALUATOR INSTRUCTIONS:

Objective: to determine the student's competency level when operating fluoroscopic equipment.
 The student will be able to:

FLUOROSCOPY EQUIPMENT				
Yes	No	N/A	1	Demonstrate locking the fluoroscopic tower over the table.
Yes	No	N/A	2	Demonstrate angulation of the table.
Yes	No	N/A	3	Move the table both longitudinally and transversely using the controls on the table.
Yes	No	N/A	4	Move the table longitudinally and transversely using the controls on the tower.
Yes	No	N/A	5	Load and unload spot films in the tower.
Yes	No	N/A	6	Program the machine for full, split, horizontal, and four-on-one spot films.
Yes	No	N/A	7	Demonstrate how to activate the grid devise.
Yes	No	N/A	8	Demonstrate how to activate the compression device.
Yes	No	N/A	9	Move the tower the length of the table using the motor driven handle.
Yes	No	N/A	10	Demonstrate magnification.
Yes	No	N/A	11	Demonstrate collimation.
Yes	No	N/A	12	Demonstrate image reversal.
Yes	No	N/A	13	Demonstrate resetting cumulative and five minute timer.
Yes	No	N/A	14	Identify controls: Set fluoroscopy settings as appropriate to equipment.
DIGITAL FLUOROSCOPY				
Yes	No	N/A	15	Using the computer, add a new patient's information.
Yes	No	N/A	16	Select images for printing or transfer to PACS.
Yes	No	N/A	17	Manipulate any image to include changes to: the scale of contrast, brightness, zoom or magnify, invert etc.
Yes	No	N/A	18	Select print style based on radiologist preference. Negative vs positive.
Yes	No	N/A	19	Print selected images.
Yes	No	N/A	20	Retrieve a previous patient's images to the screen.

COMMENTS: